



REDLINE is a unique, self-testing, disposable, simple, inexpensive and laboratory calibrated alcohol breathalyser, which detects the presence of alcohol in the body and gives an accurate indication of whether the blood alcohol content has exceeded a specific limit.

Redline is the ONLY disposable breathalyser in the world to hold French, German and Australian Government approvals for both quality of performance and manufacturing standards.

Its low cost, accuracy and ease of use make the REDLINE testers ideal screening tools for:

PARENTS: Checking for teenage alcohol consumption

DRIVING SCHOOLS: Checking students (or instructors) for the presence of alcohol

MOTORSPORT: Promoters/organisers to check for the presence of alcohol amongst competitors

BUS/TRUCK COMPANIES: Checking drivers for the presence of alcohol before they depart

AVIATION: Checking pilots, cabin crew or ground-staff for the presence of alcohol

COMPANIES: Checking for alcohol consumption by workers, especially if workers are using vehicles, machinery, hazardous goods, power tools, working in hazardous areas, working at height, working with electricity/explosives/flammable goods or any other safety risk.

SCHOOLS/COLLEGES/TAFE/UNIVERSITIES: Checking students for the presence of alcohol

MINING INDUSTRY: REDLINE provides a non-invasive "first screening" with results available in 2 minutes. This then qualifies an individual for submission to a forensic-quality evidential test.

HOW TO USE:

Wait 15 minutes after your last alcoholic drink.

- The user must break off the snap-off end caps at both ends of the tester and shake out the harmless white crystals inside the tube.
- The plastic measuring bag must then be inflated by blowing in one steady breath into the neck of the bag.
- The valve in the neck of the bag automatically closes after inflation.
- The tester is then inserted into the neck of the bag and the contents of the bag squeezed through the tester.
- The presence of alcohol is indicated by a distinctive green color change.
- If the green color extends beyond the RED LINE calibrated on the tube, then it indicates a blood alcohol content of more than the calibrated level.

By measuring the alcohol content in the breath, a reliable indication of the blood alcohol level is achieved. The Redline tester can be used once and then disposed of, minimizing contamination associated with repeated use of non-disposable units.

ADVANTAGES:

Quick and Accurate	Affordable (less than \$4.00 per test)
Portable and Disposable	Unlimited Shelf Life
Requires no batteries or power source	Easy to use
A new test is conducted each time	No residual alcohol left in tester

APPROVALS:

Redline testers are certified to the French NF 227 special regulations and Standards Australia. Plus approved by:

The NF of France for both the Public, Police and Gendamerie, TUV Rheinland of Germany, Schuster Laboratories of the USA, ESR Testing Laboratories of New Zealand, Spanish National Testing Laboratories, Austrian and French Automobile Associations, Securite Routiere and the Prevention Routiere, Touring Club of Switzerland, Arrive Alive South Africa

Order Form:

School/Organisation: _____ PRIVATE
Contact Person: _____ Phone: (____) _____ Email: _____
Delivery Address: _____ Postcode: _____

REDLINE ALCOHOL DETECTORS	#		PRICE	SUBTOTAL
10 test units + (1 set) instructions		@	\$44.00	
50 test units + (50 sets) instructions		@	\$209.00	
100 test units + (100 sets) instructions		@	\$396.00	
Postage & Handling				\$ 3.50
TOTAL AMOUNT PAYABLE				\$.

Prices include GST. Payment or Purchase Order must be received in advance. Discounts apply for large orders
Prices may change without notice. Make all cheques payable to **Safe Drive Training (Aust) Pty Ltd.**
Please allow 2 weeks for delivery. Prices may vary for locations outside Australia.

CREDIT CARD PAYMENTS: MasterCard Visa

Card Number

Expiry ___/___ Cardholder's Name _____ Signature _____

REDLINE IS SOLD UNDER THE FOLLOWING TERMS AND CONDITIONS:

READ THE USER INSTRUCTIONS CAREFULLY. Accuracy of test results may not be reliable if test is not conducted according to instructions. Refrain from driving after drinking any alcohol. Do not drive if the crystals change colour. The manufacturers, suppliers, agents, distributors and retailers make no warranty, expressed or implied, as to the ability of this device to determine or detect intoxication of the subject or to accurately indicate the subject's blood alcohol level. Decisions and/or actions based on the use of this product by any person shall be at such person's own risk. The manufacturers, suppliers, agents, distributors and retailers assume no responsibility for consequences of subjects who test negative using this device but who later show that they are under the influence of or their judgement has been impaired by alcohol.

**POSTAL ADDRESS: P O BOX 3747 LOGANHOLME HYPERDOME QLD 4129
PHONE: (07) 3801 3222 FAX: (07) 3801 5455 EMAIL: info@sdt.com.au**